Canine *Angiostrongylus vasorum*
Antigen Test Kit

For veterinary use only.

Angio Detect® Test

In vitro diagnostic for the detection of *Angiostrongylus vasorum* antigen in canine serum or plasma. The presence of this antigen in canine blood indicates the animal is actively infected with the *A. vasorum* parasite.

Precautions and warnings.

- All wastes should be properly decontaminated prior to disposal.

Storage



- Refrigeration is not required.
- Store test kits at 4–30°C until expiration date.
- Use test within 30 minutes after opening the sealed pouch.

Kit components

C = Control T = Test S = Circular sample well

Item	Reagent	Quantity
1	One bottle of buffer	4.0 ml
2	Test device	5 or 20

Other components: Transfer pipettes

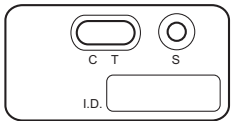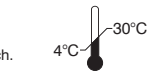Sample information

- Samples must be at 18–25°C before beginning the test procedure.
- Serum or plasma, either fresh or stored at 2–8°C for up to one week, can be used.
- For longer storage, serum or plasma can be frozen (–20°C or colder) and then recentrifuged before use.
- Hemolyzed or lipaemic samples will not affect test results.

Test procedure

1

Remove the test device from the sealed pouch and place the device on a flat horizontal surface.

2

Using the pipette provided, dispense **3 drops of sample** into the circular sample well (S). Allow the sample to be completely absorbed.

3

After the sample is completely absorbed, remove the blue cap only from the test buffer bottle, leaving the white cap firmly attached. Then hold the buffer bottle vertical and **add 3 drops of buffer** to the circular sample well.

4

The sample and buffer will flow across the result window.

5

Read test results at 15 minutes.

Interpreting the test results

Positive result

Any pink/purple colour development on the control line (C) and the test line (T) is a positive result.

Negative result

Pink/purple colour development on the control line (C) only is a negative result.

Invalid results

If the control line (C) does not develop, the results are invalid.

Note: The control line (C) may develop sooner, but results are not complete until 15 minutes have elapsed.

Sensitivity and specificity

Comparison test	Sample size Angio Detect/reference test ^a					Sample type	Relative sensitivity and specificity 95% Confidence limit
	+/+	-/+	+/-	-/-	Total		
<i>Angiostrongylus vasorum</i> ^a	53	1 ^b	2 ^c	174	230	Serum/ plasma	Sen., 98.1% (95% CL 88.9%–99.9%) Spec., 99.4% (95% CL 95.5%–99.9%)
^a Baermann Faecal Flotation, ^b Negative by Zurich ELISA, ^c Positive by Zurich ELISA							

Sensitivity and specificity are based on visual interpretation of Angio Detect Test results.

CL = Confidence limit

Symbol descriptions

	Use by date		Authorised representative in the European Community
	Batch code (Lot)		
	Catalogue number		Consult instructions for use
	Temperature limitation		In vitro diagnostic
	Manufacturer		

IDEXX Technical Support
Europe: idexx.eu

IDEXX

*Angio Detect is a trademark or registered trademark of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

© 2025 IDEXX Laboratories, Inc. All rights reserved.

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