# **Amylase**

# Interpretive Summary

**Description:** Amylase is an enzyme produced primarily in the pancreas to digest dietary carbohydrates in the intestine.

# **Decreased Amylase**

#### **Common Causes**

- Severe hepatic dysfunction/portosystemic shunts
- Decreased amylase is seldom clinically significant

## **Increased Amylase**

### **Common Causes**

- Pancreatic acinar cell damage
  - o Inflammation
  - o Necrosis
  - Neoplasia
  - o Pancreatic duct obstruction
- Decreased renal clearance/renal inactivation (any condition that reduces glomerular filtration rate [GFR])
  - o Pre renal disorders
    - Shock
    - Dehydration
    - Poor cardiac output
  - Renal disease
  - Post-renal disorders
    - Urinary tract obstruction or rupture

#### **Uncommon Causes**

- Enteritis
- Hepatobiliary disease
- Macroamylasemia
  - o Amylase is complexed to immunoglobulins/other proteins too large to pass through the glomerulus
  - o Extends the half life of amylase in the circulation

# Related findings

- Pancreatic acinar cell damage
  - o Increased lipase
  - Increased Spec cPL® /Spec fPL®
  - o Increased ALP +/- increased bilirubin
  - o Increased glucose
  - Inflammatory leukogram on CBC
- Decreased renal clearance/renal inactivation
  - Pre renal
    - Increased hematocrit and total protein
    - Increased BUN, creatinine, and possibly phosphorous
    - Increased urine specific gravity
    - Decreased blood pressure (shock, poor cardiac output)
  - Renal



- Increased BUN, creatinine, phosphorus
- Isosthenuria
- Post renal
  - Increased BUN, creatinine, and possibly phosphorus
  - Abnormalities on urinalysis (blood, inflammation, crystals, etc.)
  - Variable urine specific gravity
  - Abnormalities associated with urinary tract obstruction/rupture on abdominal ultrasound or contrast studies

### **Additional Information**

## **Physiology**

- Amylase is a cytoplasmic enzyme produced by the pancreas that catalyzes the hydrolysis of complex starches in the intestine.
- Although one of the main sources of amylase is the pancreas, it is produced from many other tissues in the body, making it a relatively nonspecific indication of pancreatic inflammation.
- Sources of amylase activity in blood include pancreas, intestine, macroamylases (protein complexes containing amylase bound to other proteins), and possibly liver.
- Serum amylase activity peaks at 12-48 hours and can return to the reference interval within 8-14 days after a single pancreatic insult.
- The kidneys excrete or inactivate amylase. Conditions that reduce GFR usually do not result in more than a 3 fold increase in serum amylase activity.
- In dogs, the higher the serum amylase activity (3 to 4 fold increase), the more likely the presence of pancreatic disease. Serum amylase activity can be increased 7 to 10 fold above the reference interval with pancreatic disease in this species.
- In cats, serum amylase activity is variable in patients with pancreatitis and can be decreased (experimental cases), within reference intervals, or mildly increased.
- In horses serum amylase activity is not diagnostically useful.

### **Diagnostic Methodology**

Assays measure amylase activity through substrate consumption, not the quantity of circulating amylase.

#### References

- Ettinger SJ, Feldman EC, eds. *Textbook of Veterinary Internal Medicine*, 6th ed. St. Louis, MO: Elsevier Saunders; 2005.
- Latimer KS, Mahaffey EA, Prasse KW, eds. *Duncan and Prasse's Veterinary Laboratory Medicine: Clinical Pathology*, 4th ed. Ames, IA: Blackwell; 2003
- Stockham SL, Scott MA. Fundamentals of Veterinary Clinical Pathology, 2nd ed. Ames, IA: Blackwell; 2008.
- Tams, TR. Handbook of Small Animal Gastroenterology, 2nd ed. St. Louis, MO: Saunders; 2003.
- Willard MD, Tvedten H, eds. *Small Animal Clinical Diagnosis by Laboratory Methods*, 4th ed. St. Louis, MO: Saunders; 2004.

Last updated 11/1/2013

