

SNAP FIV/FeLV Combo Test

Test accuracy

Sensitivity and specificity of the SNAP® FIV/FeLV Combo Test

Analyte	Reference method	Device result		Total	Sample type	Sensitivity (95% CL)
		+	-			Specificity (95% CL)
FeLV ¹	+	72	1	237	Serum/plasma/ whole blood	98.6% (91.8%–100%)
	-	3	161			98.2% (94.5%–99.6%)
FIV ²	+	43	3	237	Serum/plasma/ whole blood	93.5% (81.7%–98.3%)
	-	0	191			100% (97.6%–100%)

Sensitivity and specificity are based on visual interpretation of SNAP® test results.

CL = confidence limit

Reference Method:

1. Petchek® FeLV Antigen Test
2. Petchek® FIV Antibody Test

For retrovirus test interpretation and diagnostic algorithms, please refer to AAFP Feline Retrovirus Management guidelines, available at catvets.com/guidelines